

Instructions for Use: Silicone Finger Mats

Brand Name of Product	Silicone Finger Mats
Generic Name of Product	Silicone Finger Mats
Product Code Number(s)	2-2025, 2-1405, 2-3215, 2-1055, 2-1215, 6-1914, 2-4005, 2111-M, 2-6005, SR10, SP2313, SP4622, SP2719
Intended Use	To protect delicate items to hold in place and prevent damage.
Range of Applications for Product	Designed to maximize space for delicate items and accessories.
Key Specifications of Product	<ul style="list-style-type: none"> • Material: Silicone • Various sizes • Reusable.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	Store in clean, dry conditions on shelf in original packaging.
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	To protect delicate items to hold in place and prevent damage.
Preparation for Use	<ul style="list-style-type: none"> • Inspect the Silicone Finger Mats before processing. • Clean prior to first use in accordance with the cleaning instructions. • Withdraw damaged mats and report to manager. • Rinse with warm water to remove any residues.
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ol style="list-style-type: none"> 1. With fingers pointed upwards, place the appropriate size Silicone Finger Mat in tray. 2. Place delicate items in a single layer and with adequate spacing to prevent contacting each other. 3. Place lid onto tray to secure items.
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> • Do not use abrasive powders or metal brushes, as they may cause surface damage. • Do not exceed 137 °C (279 °F). • Do not exceed eight (8) minutes.
Disposal	<ul style="list-style-type: none"> • Discard mat with visible wear and tear or damage to the mat. • Dispose of in accordance with clinical waste procedures.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	<ol style="list-style-type: none"> 1. Wash mats with detergents, and disinfectants with a range between 2 pH and 13 pH; enzymatic or biocide cleaners compatible with polypropylene material (see detergent Mfr's guide). 2. Final Rinse with AAMI Critical Water.
Cleaning – Automated	<ul style="list-style-type: none"> • Place mat in automatic washer outside of tray. • Run "Instrument Cycle" of the Washer.
Disinfection	Mats can be disinfected with the thermal disinfection stage of the Washer-Disinfectors
Drying	Dry with non-linting wipe.
Maintenance, Inspection, and Testing	<ul style="list-style-type: none"> • Visually inspect product to identify damage. • Discard damaged mats.

Reassembly Instructions	N/A
Packaging	N/A
Sterilization	Material compatible with Steam, EO, and VH2O2 Sterilization. Recommended cycle for steam sterilization (up to 1000 cycles): <ul style="list-style-type: none"> • 132 °C (270 °F) for four (4) minutes • 135 °C (275 °F) for three (3) minutes.
Storage	N/A
Additional Information	N/A
Related Healthmark Products	N/A
Other Product Support Documents	Instrument Trays Brochure, Instrument Trays Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com