

Instructions for Use: UnderGuard™ Tray Corners

Brand Name of Product	UnderGuard™ Tray Corners
Generic Name of Product	Paper Tray Corners
Product Code Number(s)	TC-001, TC-001ASM, TC-004
Purpose of Product	To protect the corner of sterilization wrap from the sharp edges of instrument trays and containers.
Range of Applications for Product	Wrapped sterilization trays that are steam sterilized.
Key Specifications of Product	<ul style="list-style-type: none"> Medical grade cellulose: TC-ASM (001 & 002) Assembled Tray Corners.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	<ul style="list-style-type: none"> Temperature: 15 °C–30 °C (59 °F–86 °F) Relative Humidity: 30–70% rh
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	To protect the corner of sterilization wrap from the sharp edges of instrument trays and containers.
Preparation for Use	N/A
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ol style="list-style-type: none"> 1. Fold. 2. Tuck. 3. Tuck. 4. Pinch. 5. Place on corners of the tray.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	Single use. Follow facility guidelines based on disposal of sterilization packaging material.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	<ul style="list-style-type: none"> Steam sterilization: <ul style="list-style-type: none"> 132 °C (270 °F) for 4 minutes 135 °C (275 °F) for 3 minutes. Gravity steam sterilization: 121 °C (250 °F) for 30 minutes. Maximum extended time: 135 °C (275 °F) for 30 minutes. Compatible with EO sterilization.
Storage	N/A
Additional Information	N/A

Related Healthmark Products	N/A
Other Product Support Documents	Sterilization Price List, Sterilization Brochure
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com