

Instructions for Use: UnderGuard™ Tray Liners

| | |
|--|---|
| Brand Name of Product | UnderGuard™ Tray Liners |
| Generic Name of Product | Paper Tray Liners |
| Product Code Number(s) | TL-0812, TL-1010, TL-1020, TL-1216, TL-1822, TL-1212, TL-1218, TL-1224, TL-1226, TL-1818, TL-1824, TL-0610, TL-0624, TL-0711, TL-0509, TL-2025, BL-1414, BL-1419, BL-1423 |
| Intended for Use | <ul style="list-style-type: none"> • Mitigate against the incidence of wet pack. <ul style="list-style-type: none"> ○ Line the bottom of trays and containers during to absorb and rapidly disperse condensate. ○ Liners can be safely used inside trays or can be used underneath trays. • Tray liners can be used as a packaging accessory. <ul style="list-style-type: none"> ○ Place inside of wrapped packages. ○ Protect containers. ○ Separate items and provide wicking. |
| Range of Applications for Product | For use in trays including wrapped and closed containers. |
| Key Specifications of Product | <ul style="list-style-type: none"> • Single use. • Non-linting. • Form fitting tray liners also prevent instruments from protruding through openings on the bottom and sides of trays and baskets. |

| Shipping & Storage | |
|---|---|
| Shipping Conditions & Requirements | N/A |
| Storage Conditions | <ul style="list-style-type: none"> • Temperature: 15 °C (59 °F) to 30 °C (86 °F) • Humidity: 30–70% rh. |
| Packaging Contents | N/A |
| Shelf Life | N/A |

| Instructions for Using Product | |
|--|---|
| Description of Use(s) | To line the bottom of trays and containers to absorb and rapidly disperse condensate to mitigate against the incidence of wet pack. |
| Preparation for Use | <ol style="list-style-type: none"> 1. Select a liner which will best fit the target basket. 2. If using with a wrapped tray, liner must be inside the wrap and may be used inside or under the basket. 3. If using with a closed container, liner must be used inside the container (either inside or below the basket). |
| Diagrams (drawings, pictures) | N/A |
| Steps for Use of Product | <ol style="list-style-type: none"> 1. Place liner either inside or beneath basket. If you are using a container the liner should be placed in case first. 2. Place items in tray. 3. Wrap or close package. |
| Interpretation of Results | N/A |
| Contraindications of Test Results | N/A |
| Documentation | N/A |
| Special Warnings and Cautions | N/A |
| Disposal | Single use. Dispose of in compliance with facility's policy. |

| Reprocessing Instructions | |
|--|-----|
| Point of Use | N/A |
| Preparation for Decontamination | N/A |
| Disassembly Instructions | N/A |
| Cleaning – Manual | N/A |
| Cleaning – Automated | N/A |
| Disinfection | N/A |
| Drying | N/A |

| | |
|---|--|
| Maintenance, Inspection, and Testing | N/A |
| Reassembly Instructions | N/A |
| Packaging | N/A |
| Sterilization | <ul style="list-style-type: none"> • Steam sterilization: <ul style="list-style-type: none"> ○ 132 °C (270 °F) for 4 minutes ○ 135 °C (275 °F) for 3 minutes. • Gravity steam sterilization at 121 °C (250 °F) for 30 minutes. • Maximum extended time for 30 minutes at 135 °C (275 °F). • Compatible with EO sterilization. |
| Storage | N/A |
| Additional Information | N/A |

| | |
|--|---|
| Related Healthmark Products | N/A |
| Other Product Support Documents | Sterilization Brochure, Sterilization Price List |
| Reference Documents | N/A |
| Customer Service Contact | Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com |