


**Instructions for Use:** UnderGuard for Plasma™

<b>Brand Name of Product</b>	UnderGuard for Plasma™
<b>Generic Name of Product</b>	Tray Liners
<b>Product Code Number(s)</b>	MT-2030, MT-1520, MT-1224
<b>Purpose of Product</b>	For use under the tray to protect outer wrap from sharp corners and imperfections of trays.
<b>Range of Applications for Product</b>	N/A
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>Manufactured from spun polypropylene.</li> <li>Unique purple color helps distinguish protective layer</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	<ul style="list-style-type: none"> <li>Temperature: 15 °C (59 °F) to 30 °C (86 °F)</li> <li>Humidity: 30–70% rh.</li> </ul>
<b>Packaging Contents</b>	N/A
<b>Shelf Life</b>	Five (5) years from manufacturer's date.

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	To protect outer wrap from sharp corners and imperfections of trays.
<b>Preparation for Use</b>	<ul style="list-style-type: none"> <li>Select a liner which will best fit the target basket.</li> <li>If using with a wrapped tray, the liner must be inside the wrap and may be used inside or under the basket.</li> <li>If using with a closed container, the liner must be used inside the basket (either inside or below the basket).</li> </ul>
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>Place a liner either inside or beneath basket. (In the event you are using a container, the liner should be placed in basket first.)</li> <li>Place items in tray.</li> <li>Wrap or close package.</li> <li>Place in VH2O2, VH2O2-O3, or steam sterilizer.</li> </ol>
<b>Interpretation of Results</b>	N/A
<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	N/A
<b>Special Warnings and Cautions</b>	N/A
<b>Disposal</b>	MT-XXXX are number 5  for recycling.

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	N/A
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A
<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	Material compatible with VH2O2/O3 and steam sterilization up to 137 °C (278.6 °F).
<b>Storage</b>	<ul style="list-style-type: none"> <li>Temperature: 15 °C (59 °F) to 30 °C (86 °F)</li> <li>Humidity: 30–70% rh.</li> </ul>
<b>Additional Information</b>	N/A

<b>Related Healthmark Products</b>	N/A
<b>Other Product Support Documents</b>	Sterilization Brochure, Sterilization Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>