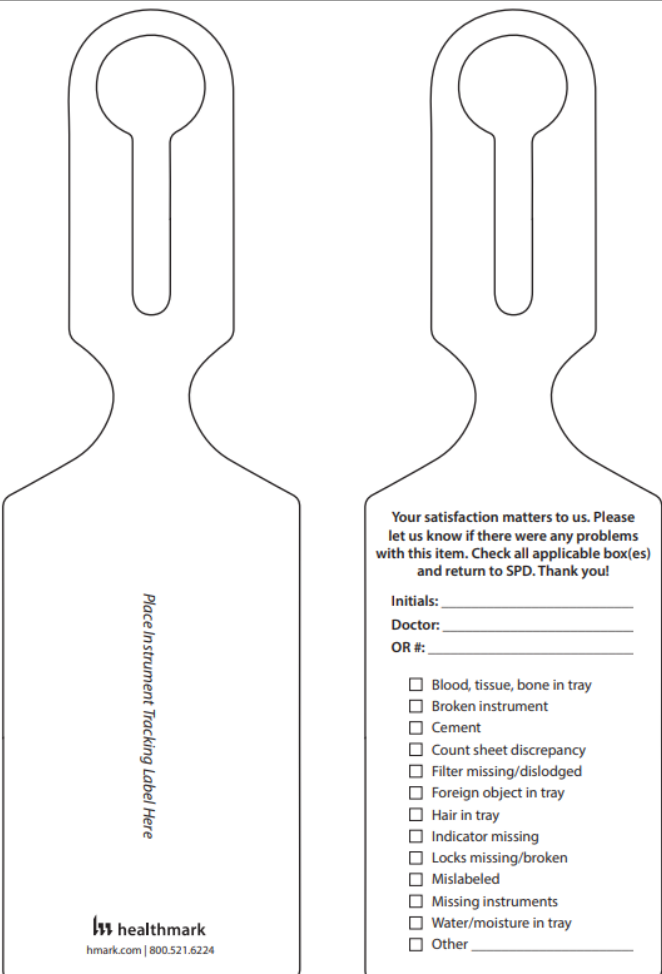


## Instructions for Use: Quality Audit Tray Label

<b>Brand Name of Product</b>	Quality Audit Tray Label
<b>Generic Name of Product</b>	Self-Looping Label
<b>Product Code Number(s)</b>	403225 QAT
<b>Intended Use</b>	Self-Looping Labels facilitate specific markings/remarks. Designed to communicate important information.
<b>Range of Applications for Product</b>	To identify any problems with items.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Self-Looping style for mounting on the handle of any closed container.</li> <li>• 8.5 L x 2.5 W inches</li> <li>• Material: White PET</li> <li>• White with black text, double-sided print.</li> </ul>

Shipping & Storage	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	N/A
<b>Packaging Contents</b>	Box of 500
<b>Shelf Life</b>	N/A

Instructions for Using Product	
<b>Description of Use(s)</b>	To identify any problems with items.
<b>Preparation for Use</b>	N/A
<b>Diagrams (drawings, pictures)</b>	 <p style="text-align: center;">Place Instrument Tracking Label Here</p> <p style="text-align: center;">healthmark hmark.com   800.521.6224</p> <p>Your satisfaction matters to us. Please let us know if there were any problems with this item. Check all applicable box(es) and return to SPD. Thank you!</p> <p>Initials: _____      Doctor: _____      OR #: _____</p> <p> <input type="checkbox"/> Blood, tissue, bone in tray  <input type="checkbox"/> Broken instrument  <input type="checkbox"/> Cement  <input type="checkbox"/> Count sheet discrepancy  <input type="checkbox"/> Filter missing/dislodged  <input type="checkbox"/> Foreign object in tray  <input type="checkbox"/> Hair in tray  <input type="checkbox"/> Indicator missing  <input type="checkbox"/> Locks missing/broken  <input type="checkbox"/> Mislabeled  <input type="checkbox"/> Missing instruments  <input type="checkbox"/> Water/moisture in tray  <input type="checkbox"/> Other _____     </p>
<b>Steps for Use of Product</b>	1. Affix an instrument tracking label to the front of the tag.

	<ol style="list-style-type: none"> <li>2. Loop the tray label through the handle of the container or tray prior to sterilization.</li> <li>3. When the tray is received in the OR, complete the Quality Audit Tray Label with: <ol style="list-style-type: none"> <li>a. User's initials</li> <li>b. Doctor's name</li> <li>c. OR room number</li> <li>d. Check all applicable boxes.</li> </ol> </li> <li>4. Return to SPD.</li> </ol>
<b>Interpretation of Test Results</b>	N/A
<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	N/A
<b>Special Warnings and Cautions</b>	<ul style="list-style-type: none"> <li>• Do not allow tray labels to be in physical contact with surgical instruments during sterilization cycles.</li> <li>• Do not handle tray labels until they cool to room temperature after sterilization or thermal disinfection.</li> </ul>
<b>Disposal</b>	Single use. Damaged labels should be disposed of in accordance with facility policy.

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	N/A
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A
<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	<ul style="list-style-type: none"> <li>• Compatible with Steam, EO, and VH2O2 sterilization.</li> <li>• Recommended cycle time for steam sterilization: <ul style="list-style-type: none"> <li>○ 132 °C (270 °F) for four (4) minutes</li> <li>○ 135 °C (275 °F) for three (3) minutes.</li> </ul> </li> </ul>
<b>Storage</b>	N/A
<b>Additional Information</b>	N/A
<b>Related Healthmark Products</b>	N/A
<b>Other Product Support Documents</b>	Labels Brochure, Labels Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>