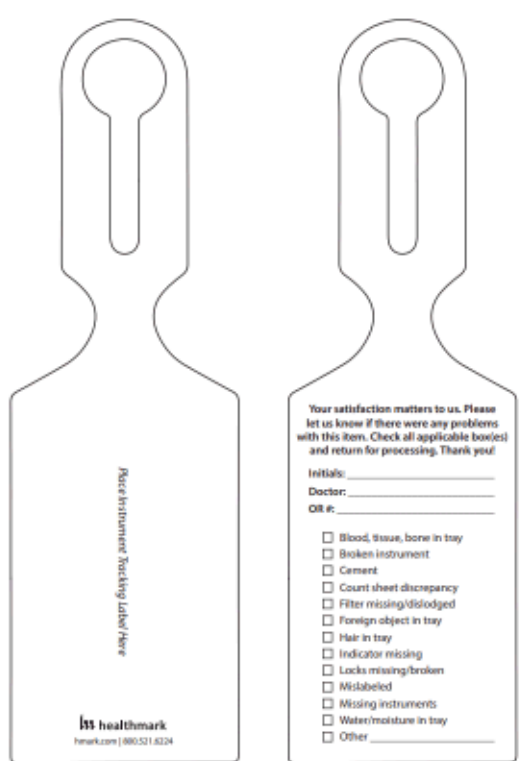


Instructions for Use: Quality Audit Tray Label

Brand Name of Product	Quality Audit Tray Label
Generic Name of Product	Self-Looping Label
Product Code Number(s)	403225 QAT
Intended Use	Self-Looping Labels facilitate specific markings/remarks. Designed to communicate important information.
Range of Applications for Product	To identify any problems with items.
Key Specifications of Product	<ul style="list-style-type: none"> • Self-Looping style for mounting on the handle of any closed container. • 8.5 L x 2.5 W inches • Material: White PET • White with black text, double-sided print.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	Box of 500
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	To identify any problems with items.
Preparation for Use	N/A
Diagrams (drawings, pictures)	 <p> Your satisfaction matters to us. Please let us know if there were any problems with this item. Check all applicable boxes and return for processing. Thank you! Initials: _____ Doctor: _____ OR #: _____ </p> <p> <input type="checkbox"/> Blood, tissue, bone in tray <input type="checkbox"/> Broken instrument <input type="checkbox"/> Cement <input type="checkbox"/> Count sheet discrepancy <input type="checkbox"/> Filter missing/dislodged <input type="checkbox"/> Foreign object in tray <input type="checkbox"/> Hair in tray <input type="checkbox"/> Indicator missing <input type="checkbox"/> Locks missing/broken <input type="checkbox"/> Mislabeled <input type="checkbox"/> Missing instruments <input type="checkbox"/> Water/moisture in tray <input type="checkbox"/> Other _____ </p>
Steps for Use of Product	<ol style="list-style-type: none"> 1. Affix an instrument tracking label to the front of the tag. 2. Loop the tray label through the handle of the container or tray prior to sterilization. 3. When the tray is received in the OR, complete the Quality Audit Tray Label with: <ol style="list-style-type: none"> a. User's initials b. Doctor's name c. OR room number

	d. Check all applicable boxes. 4. Return to SPD.
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> Do not allow tray labels to be in physical contact with surgical instruments during sterilization cycles. Do not handle tray labels until they cool to room temperature after sterilization or thermal disinfection.
Disposal	Single use. Damaged labels should be disposed of in accordance with facility policy.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	<ul style="list-style-type: none"> Compatible with Steam, EO, and VH2O2 sterilization. Recommended cycle time for steam sterilization: <ul style="list-style-type: none"> 132 °C (270 °F) for four (4) minutes 135 °C (275 °F) for three (3) minutes.
Storage	N/A
Additional Information	N/A
Related Healthmark Products	N/A
Other Product Support Documents	Labels Brochure, Labels Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com