

Instructions for Use: Utility Scissors

Brand Name of Product	Utility Scissors
Generic Name of Product	Scissors
Product Code Number(s)	FL-US
Intended Use	Designed to cut the Flexible Stem Brushes to the desired length.
Range of Applications for Product	To cut brushes to the desired length.
Key Specifications of Product	<ul style="list-style-type: none"> • Stainless-steel blade • Plastics, black handle • 7.5-inch length • Scissors are reusable.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	Sold by the each.
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	To cut the Flexible Stem Brushes to the desired length.
Preparation for Use	N/A
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place the blades of the scissors on the Flexible Stem Brush. 2. Snip the stem off to the desired length.
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	N/A

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	Clean with a neutral pH detergent.
Cleaning – Automated	<ul style="list-style-type: none"> • Machine cleaning is recommended in a washer disinfector complying with the ISO 15883 series of standards. • <i>No overlapping</i>- partially covered items will not be washed properly. • For the final rinse, AAMI Critical Water is recommended. • The standard program to include: <ol style="list-style-type: none"> 1. Pre-wash with cold water (< 38 °C [100.4 °F]), and rinse for a minimum of two (2) minutes. 2. Washing cycle with alkaline or enzymatic detergent at temperature recommended by the detergent manufacturer for a minimum of two (2) minutes. 3. Thermal disinfection of up to 93 °C (199.4 °F) in compliance with the washer manufacturer’s recommendations for time and temperature. 4. Drying cycle at a temperature not to exceed 116 °C (240.8 °F). 5. Use caution when unloading machine as products will be hot. 6. Visual inspection is required to ensure complete removal of soil. 7. If there is visible soil, repeat the cleaning program.
Disinfection	<ul style="list-style-type: none"> • The thermal disinfection stage of an automated washer is sufficient for disinfection of the utility scissors. The minimum recommended time and temperature is one (1)-minute at 82 °C (179.6 °F).

	<ul style="list-style-type: none"> Utility scissors may be disinfected with liquid chemical disinfectants in accordance with the disinfection manufacturers' IFUs. (Note: Be sure to check material compatibility information from the disinfectant manufacturer.)
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	<p>Material is compatible with Steam sterilization.</p> <ul style="list-style-type: none"> 132 °C (270 °F) for four (4) minutes. 135 °C (275 °F) for three (3) minutes. Extended cycle 135 °C (275 °F) for 30 minutes.
Storage	N/A
Additional Information	N/A
Related Healthmark Products	N/A
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	N/A
Customer Service Contact	<p>Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com</p>