


# WatchDog AI™

## Quick guide

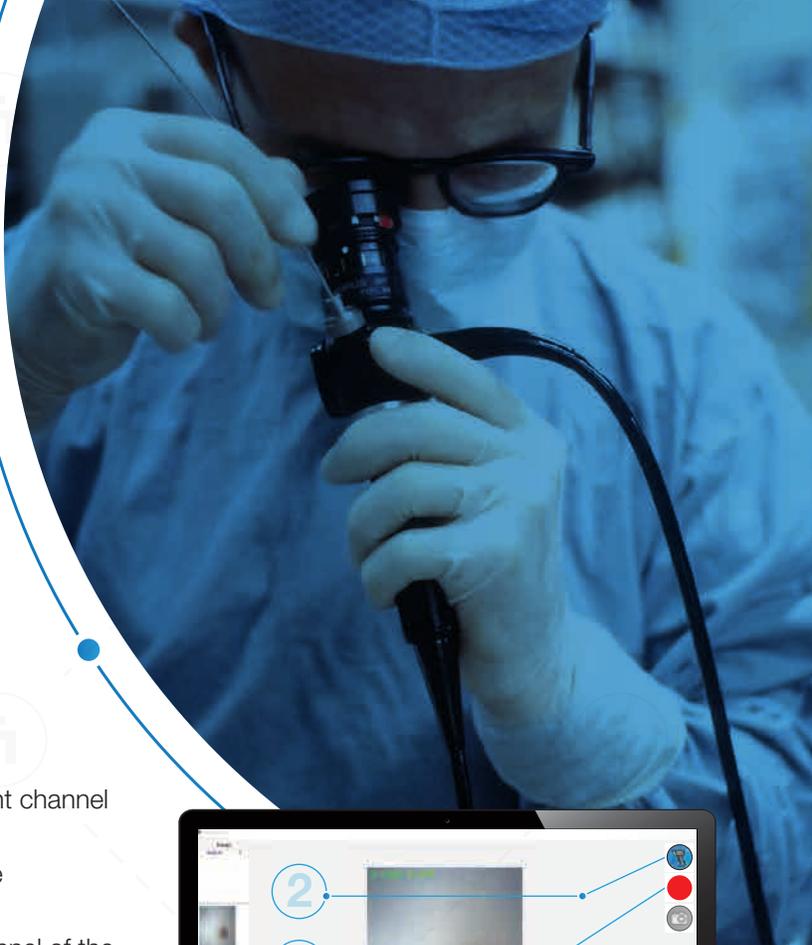

01

Log into the WatchDog AI™ inspection application.


02

Scan/enter scope I.D.

03

Inspection process

- Insert the borescope into the proximal end of the instrument channel of the endoscope.
- To activate WatchDog Artificial Intelligence press the Scope Inspection button on the user screen.
- Continue pushing the borescope down the instrument channel of the endoscope, observing the live images on the screen of the tablet or laptop.
- When inspecting longer endoscopes, it may be necessary to remove the borescope from the proximal end and insert from the distal end of the instrument channel.
- Deactivate the Scope Inspection button on the user screen.
- Total inspection time should take less than 3 minutes.
- Changing borescopes – turn off borescope at the control unit.

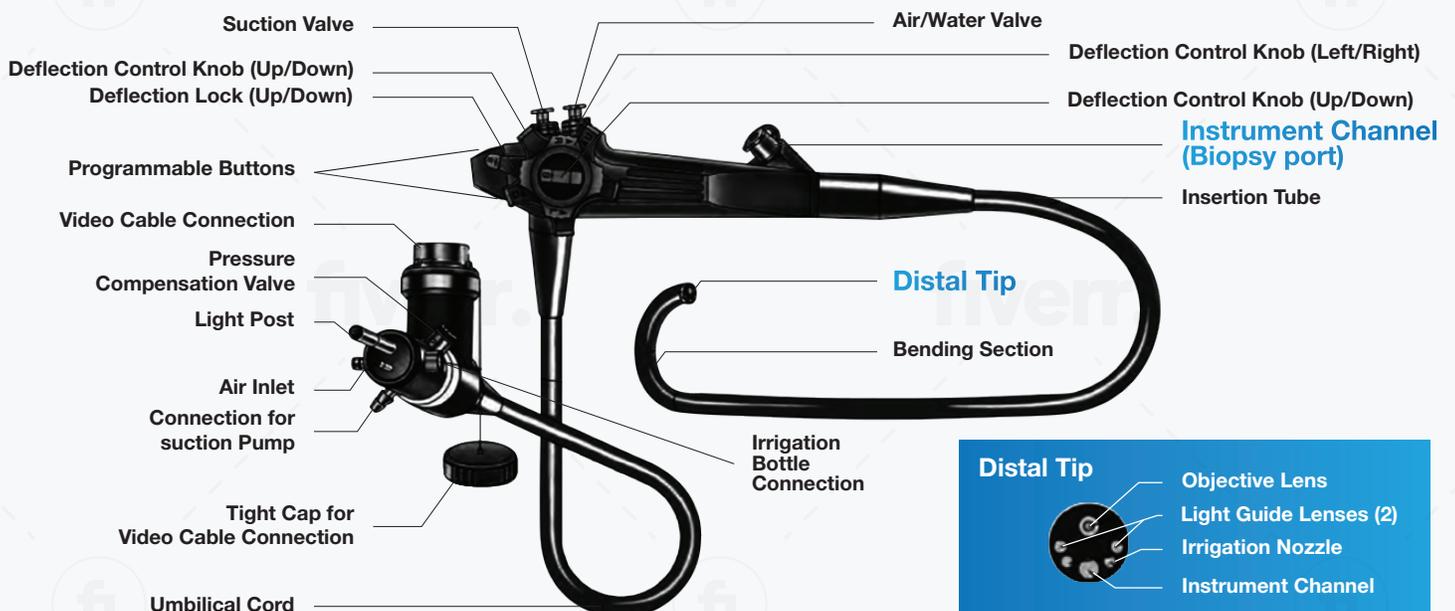



fiverr.

fiverr.

fiverr.



fiverr.

fiverr.

**WatchDog AI™**

**Contact Us**



[info@bh2innovations.com](mailto:info@bh2innovations.com)



[www.wdog.ai](http://www.wdog.ai)