

Brand Name of Product	UnderGuard™ Dry Advanced
Generic Name of Product	Dry Mats
Product Code Number(s)	DA-2030PP, DA-2030GN, DA-1520PP, DA-1520GN, DA-1224PP, DA-1224GN, DA-1212PP, DA-1020PP, DA-1020GN, DA-1010PP, DA-1010GN
Intended Use	For use inside or outside of trays within a sterile barrier. Protects items while simultaneously providing moisture wicking against the dispersion of condensate.
Range of Applications for Product	Provides protection useful for trays and baskets.
Key Specifications of Product	<ul style="list-style-type: none"> • Polypropylene • Single-use • Non-sterile.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	<ul style="list-style-type: none"> • Temperature: 10 °C (50 °F) to 30 °C (86 °F) • Humidity: 30–70% rh • Store in dry areas away from sunlight. • Not temperature controlled
Packaging Contents	N/A
Shelf Life	Five (5) years from manufacturer date.

Instructions for Using Product	
Description of Use(s)	Protects items while simultaneously providing moisture wicking against the dispersion of condensate.
Preparation for Use	N/A
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	Liners may be placed either under the tray or inside the bottom of the tray with either the white side or green side facing up.
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	<ul style="list-style-type: none"> • Single-use. • Dispose of according to facility's policy.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	<ul style="list-style-type: none"> • Steam sterilization at: <ul style="list-style-type: none"> ○ 132 °C (270 °F) for 4 minutes ○ 135 °C (275 °F) for 3 minutes. • Gravity steam sterilization at 121 °C (250 °F) for 30 minutes. • Maximum tested time/temperature: 135 °C (275 °F). for 30 minutes. • Materially compatible with VH202, and EO sterilization.
Storage	<ul style="list-style-type: none"> • Temperature: 10 °C (50 °F) to 30 °C (86 °F)

	<ul style="list-style-type: none"> • Humidity: 30–70% rh • Store in dry areas away from sunlight. • Not temperature controlled.
Additional Information	N/A
Related Healthmark Products	Dual UnderGuard™ Dry Mats
Other Product Support Documents	Sterilization Brochure, Sterilization Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com