

<b>Brand Name of Product</b>	UnderGuard™ Dry Advanced
<b>Generic Name of Product</b>	Dry Mats
<b>Product Code Number(s)</b>	DA-2030PP, DA-2030GN, DA-1520PP, DA-1520GN, DA-1224PP, DA-1224GN, DA-1212PP, DA-1020PP, DA-1020GN, DA-1010PP, DA-1010GN
<b>Intended Use</b>	For use inside or outside of trays within a sterile barrier. Protects items while simultaneously providing moisture wicking against the dispersion of condensate.
<b>Range of Applications for Product</b>	Provides protection useful for trays and baskets.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Polypropylene</li> <li>• Single-use</li> <li>• Non-sterile.</li> </ul>

Shipping & Storage	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	<ul style="list-style-type: none"> <li>• Temperature: 10 °C (50 °F) to 35 °C (95 °F)</li> <li>• Humidity: 20–70% rh</li> <li>• Store in dry areas away from sunlight.</li> <li>• Not temperature controlled</li> </ul>
<b>Packaging Contents</b>	N/A
<b>Shelf Life</b>	Five (5) years from manufacturer date.

Instructions for Using Product	
<b>Description of Use(s)</b>	Protects items while simultaneously providing moisture wicking against the dispersion of condensate.
<b>Preparation for Use</b>	N/A
<b>Diagrams (drawings, pictures)</b>	N/A
<b>Steps for Use of Product</b>	Liners may be placed either under the tray or inside the bottom of the tray with either the white side or green side facing up.
<b>Interpretation of Test Results</b>	N/A
<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	N/A
<b>Special Warnings and Cautions</b>	N/A
<b>Disposal</b>	<ul style="list-style-type: none"> <li>• Single-use.</li> <li>• Dispose of according to facility's policy.</li> </ul>

Reprocessing Instructions	
<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	N/A
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A
<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	<ul style="list-style-type: none"> <li>• Steam sterilization at: <ul style="list-style-type: none"> <li>○ 132 °C (270 °F) for 4 minutes</li> <li>○ 135 °C (275 °F) for 3 minutes.</li> </ul> </li> <li>• Gravity steam sterilization at 121 °C (250 °F) for 30 minutes.</li> <li>• Maximum tested time/temperature: 135 °C (275 °F). for 30 minutes.</li> <li>• Materially compatible with VH202, and EO sterilization.</li> </ul>
<b>Storage</b>	<ul style="list-style-type: none"> <li>• Temperature: 10 °C (50 °F) to 35 °C (95 °F)</li> </ul>

	<ul style="list-style-type: none"> <li>• Humidity: 20–70% rh</li> <li>• Store in dry areas away from sunlight.</li> <li>• Not temperature controlled.</li> </ul>
<b>Additional Information</b>	N/A
<b>Related Healthmark Products</b>	Dual UnderGuard™ Dry Mats
<b>Other Product Support Documents</b>	Sterilization Brochure, Sterilization Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com