

<b>Brand Name of Product</b>	UnderGuard™ Tray Liners/Form-Fitting
<b>Generic Name of Product</b>	Paper Tray Liners
<b>Product Code Number(s)</b>	TL-0812, TL-1010, TL-1020, TL-1216, TL-1822, TL-1212, TL-1218, TL-1224, TL-1226, TL-1818, TL-1824, TL-0610, TL-0624, TL-0711, TL-0509, TL-2025, BL-1414, BL-1419, BL-1423
<b>Intended for Use</b>	<ul style="list-style-type: none"> <li>Mitigate against the incidence of wet pack. <ul style="list-style-type: none"> <li>Line the bottom of trays and containers during sterilization to absorb and rapidly disperse condensate.</li> <li>Liners can be safely used inside or underneath trays.</li> </ul> </li> <li>Tray liners can be used as a packaging accessory. <ul style="list-style-type: none"> <li>Place inside of wrapped packages.</li> <li>Protect containers.</li> <li>Separate items and provide wicking.</li> </ul> </li> </ul>
<b>Range of Applications for Product</b>	For use in trays, including wrapped and closed containers.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>Single use.</li> <li>Non-linting.</li> <li>Form-fitting tray liners also prevent instruments from protruding through openings on the bottom and sides of trays and baskets.</li> </ul>

Shipping & Storage	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	<ul style="list-style-type: none"> <li>Temperature: 15 °C (59 °F) to 30 °C (86 °F)</li> <li>Humidity: 30–70% rh.</li> </ul>
<b>Packaging Contents</b>	N/A
<b>Shelf Life</b>	N/A

Instructions for Using Product	
<b>Description of Use(s)</b>	Line the bottom of trays and containers to absorb and rapidly disperse condensate to mitigate against the incidence of wet pack.
<b>Preparation for Use</b>	<ol style="list-style-type: none"> <li>Select a liner which will best fit the target basket.</li> <li><i>Wrapped tray-</i> liner must be inside the wrap and may be used inside or under the basket.</li> <li><i>Closed container-</i> liner must be used inside the container (either inside or below the basket).</li> </ol>
<b>Diagrams (drawings, pictures)</b>	N/A
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>Place liner either inside or beneath basket. (Note: If you are using a container the liner should be placed in case first.)</li> <li>Place items in tray.</li> <li>Wrap or close package.</li> </ol>
<b>Interpretation of Results</b>	N/A
<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	N/A
<b>Special Warnings and Cautions</b>	N/A
<b>Disposal</b>	Single use. Dispose of in compliance with facility's policy.

Reprocessing Instructions	
<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	N/A
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A

<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	<ul style="list-style-type: none"> <li>• Steam sterilization: <ul style="list-style-type: none"> <li>○ 132 °C (270 °F) for 4 minutes</li> <li>○ 135 °C (275 °F) for 3 minutes.</li> </ul> </li> <li>• Gravity steam sterilization at 121 °C (250 °F) for 30 minutes.</li> <li>• Maximum extended time for 30 minutes at 135 °C (275 °F).</li> <li>• Compatible with EO sterilization.</li> </ul>
<b>Storage</b>	N/A
<b>Additional Information</b>	N/A

<b>Related Healthmark Products</b>	N/A
<b>Other Product Support Documents</b>	Sterilization Brochure, Sterilization Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com