

Brand Name of Product	HushKarts
Generic Name of Product	Transport carts
Product Code Number(s)	QCT-3301, QCT-225, QCT-2301, QCT-235
Intended Use	Designed for quiet mobility, and for use as surgical case carts, CSD, Pharmacy, and Respiratory.
Range of Applications for Product	For use as surgical case carts, CSD, Pharmacy, and Respiratory.
Key Specifications of Product	<ul style="list-style-type: none"> • One (1)-piece, double-wall Polyethylene. • Rubber cushioned 4-inch bearing casters. • Slip resistance surface. • Raised sides on top. • HushKarts are washable. • Available colors: <ul style="list-style-type: none"> ○ Slate Blue ○ Dark Brown ○ Coffee Beige ○ Dusty Rose ○ Brick Red

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	For use as surgical case carts, CSD, Pharmacy, and Respiratory departments.
Preparation for Use	N/A
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ul style="list-style-type: none"> • Place “Clean” SST Trays inside for delivery to the OR and/or other clinical area. • Pick up “Dirty” containers at the same time and deliver to Decontam.
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	Do not use a metal-bristle brush.
Disposal	N/A

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	<ul style="list-style-type: none"> • May be cleaned with a steam gun or scrubbed with a lint-free towel, sponge, or nylon-bristle brush. (NOTE: Never use a metal-bristle brush) • Use a cleaning solution comprised of a non-abrasive, neutral pH detergent and water at the temperature recommended in the detergent manufacturer’s IFU. • Rinse thoroughly with critical water.
Cleaning – Automated	<ul style="list-style-type: none"> • Place HushKarts in the cart washer on cart wash cycle. • Consult and follow IFU from the washer manufacturer. • Cycle Settings: <ol style="list-style-type: none"> 1. Two (2)-minute rinse with critical water from the facility cold water source. 2. Detergent wash with neutral pH detergent. Follow the IFU from the detergent manufacturer.

	3. Thermal disinfection rinse at 82 °C (180 °F) for 1-minute.
Disinfection	Use a facility approved wipe compatible with polyethylene.
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A
Additional Information	N/A
Related Healthmark Products	SST Transport Carts
Other Product Support Documents	SST Brochure, SST Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com