

# Humipak for Robotic Instruments

Humipak® prevents organics from drying over an extended period of time to keep them ready for effective cleaning

Humipak provides a humid environment, delaying the drying of contaminants during transport - prior to cleaning, disinfection, reprocessing or sterilization. Its highly absorbent, blue viscose inner liner is sandwiched between two layers of waterproof polyester-polypropylene film.

**hmark.com**  
**800.521.6224**



Create a watertight, moist atmosphere in 3 easy steps:

- 1** Simply place the instruments inside the Humipak
- 2** Add the specified amount of water to the absorbent layer
- 3** Seal with the peel away adhesive strip

Item Number	Style	Length	Width	Qty
HPSSR01-25	Holds 1 Instrument	8.30"	32.00"	25
HPSSR01-100	Holds 1 Instrument	8.30"	32.00"	100
HPSSR04-25	Holds 4 Instruments	16.50"	32.00"	25
HPSSR04-100	Holds 4 Instruments	16.50"	32.00"	100

