


Brand Name of Product	Distal Soak Tube
Generic Name of Product	Distal tips of instrumentation
Product Code Number(s)	DS-001
Intended Use	The Distal Soak is used during the post-procedure transport. The soak tube keeps the distal end of instrumentation moist and facilitates reprocessing (preventing residuals from drying). Furthermore, the delicate tips are protected during transport.
Range of Applications for Product	Maintaining a moist state and protecting distal tips of instrumentation during transport.
Key Specifications of Product	<ul style="list-style-type: none"> • Oriented Polypropylene (OPP) tubes • Silicone cap

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Conditions	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Keeps the distal end of instrumentation moist and facilitates reprocessing.
Diagrams (drawings, pictures)	
Steps for Use of Product	<p>Applying the Distal Soak Tube</p> <ol style="list-style-type: none"> 1. After the procedure, take the Distal Soak Tube out of the prepackaging unit. 2. Fill the tube with a solution that complies with the item's IFU. (Note: The maximum volume of the tube is 20 mL.) 3. Push the distal end of the instrument through the closed hole in the cap. 4. Transport the device to the processing area. 5. When ready to begin further reprocessing, remove from the Distal Soak Tube.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> • Consult the item manufacturer's IFU for proper precleaning procedures. • ⚠ Wear appropriate PPE in compliance with the facility's policy. • ☠ Treat as biohazardous after use.
Disposal	<ul style="list-style-type: none"> • Single use, • Treat used Distal Soak Tube as a ☠ biohazard and dispose of in accordance with facility policy for contaminated items.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A

Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A
Additional Information	N/A

Related Healthmark Products	N/A
Other Product Support Documents	SST Instrument Retrieval Brochure, SST Instrument Retrieval Price List
Reference Documents	N/A
Customer Service Contact	Healthmark, A Getinge company 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com