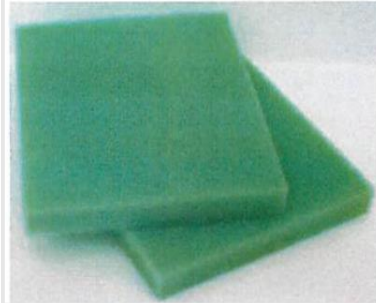



Brand Name of Product	DuraSponge™
Generic Name of Product	Cleaning Sponge
Product Code Number(s)	ESD905201, ESD904101, TCS-100
Purpose of Product	To allow for effective cleaning of sensitive items, fixtures, and all general surfaces. The tubular sponge is designed for cleaning the external surfaces of tubular items.
Range of Applications for Product	An all-purpose sponge for effective cleaning of any item requiring removal of soils.
Key Specifications of Product	<ul style="list-style-type: none"> • Retains 15% more cleaning solution than standard sponges or paper wipes. • Single use • Latex free • Nonsterile • Absorbent foam • Two (2) Styles: <ul style="list-style-type: none"> ○ <i>Multi-Use-Flat-Dry</i>: 4- W x 5 inches H ○ <i>Tubular-Dry</i>: 4.50- L x 2.50 inches D.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	Store at room temperature away from sunlight.
Packaging Contents	<ul style="list-style-type: none"> • Multi-Use-Flat-Dry: <ul style="list-style-type: none"> ○ Bulked packaged ○ 200ct/1-case. • Tubular-Dry: <ul style="list-style-type: none"> ○ Bulk individually packaged ○ 100ct/1-case.
Shelf Life	Five (5) years from date of manufacture.

Instructions for Using Product	
Description of Use(s)	For effective cleaning of sensitive items, fixtures, and all general surfaces. Use the Tubular sponge to clean the external surfaces of tubular items.
Preparation for Use	<ul style="list-style-type: none"> • Consult and follow closely the item manufacturer's (Mfr.'s) IFU. • If utilizing a cleaning agent, closely follow the detergent Mfr.'s IFU for proper preparation of the cleaning solution.
Diagrams (drawings, pictures)	<div style="display: flex; justify-content: space-around; align-items: center;"> <div style="text-align: center;">  <p>Multi-Use-Flat-Dry</p> </div> <div style="text-align: center;">  <p>Tubular-Dry</p> </div> </div>
Steps for Use of Product	<ol style="list-style-type: none"> 1. Moisten sponge with cleaning solution. 2. Wipe down the item to remove gross soil and debris.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> • Be sure to consult the item Mfr.'s IFU to ensure full compliance. • ☠ Treat used DuraSponge™ as biohazardous.

	<ul style="list-style-type: none"> • Single use only.
Disposal	Treat the used DuraSponge™ as biohazardous and discard in compliance with your facility's guidelines.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	Store at room temperature away from sunlight.
Additional Information	Manufactured by Medtrica®

Related Healthmark Products	N/A
Other Product Support Documents	ProSys™ Brochure
Reference Documents	N/A
Customer Service Contact	Healthmark, A Getinge company 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com