


Getinge Assured

Steam Tape

Getinge Assured Steam Tapes can be used in 121°C (250°F) gravity sterilizers for 30 minutes or pre-vacuum sterilizers operating at 132°C (270°F) for 4 minutes and 135°C (275°F) for 3 minutes. The stripes turn to black when exposed to steam sterilization conditions, thus providing identification of the processed items.

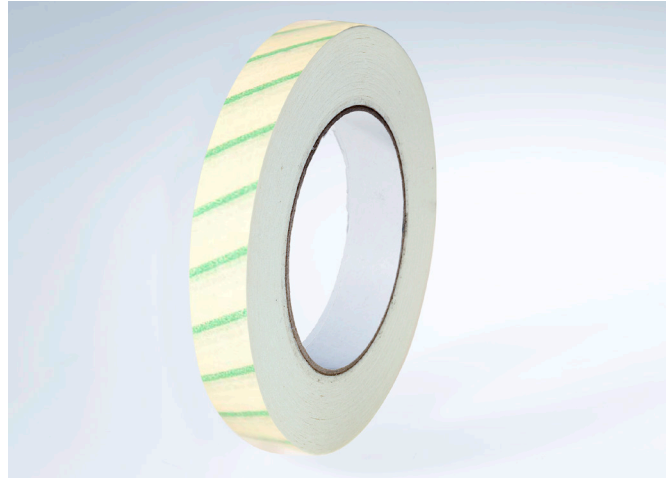

Key Features

- Not made with lead - does not require hazardous waste disposal
- Versatile tape designed for use on wraps
- Provides a secure seal and is easy to remove from reusable wraps
- Low cost per use offers economical way to monitor sterilization exposure

Performance Characteristics

Color Change: Indicator stripes turn light green to black

Physical Characteristics

- **Dimensions:** 60 yard rolls. Comes in 0.75" and 1" widths
- **Units/Case:** 0.75" = 48 rolls
1" = 36 rolls
- **Selling Unit:** Case

Storage

The storage conditions are printed on the product packaging.

Shelf Life

36 months from date of manufacture. The expiry date is printed on the product packaging

Standards Classification

- FDA Sterilization Process Indicator
- ANSI/AAMI ST79, Type 1
- ISO 11140-1 Type 1

Ordering Information

Getinge Assured

3/4" Steam Tape

1" Steam Tape

Order Number

61301606668

61301606669

Getinge is a registered trademark of Getinge AB or its subsidiaries or affiliates in the United States or other countries. • Copyright 2017 Getinge AB or its subsidiaries or affiliates. • All rights reserved. • MCV00054074 REVA

Manufactured by: SteriTec Products, Inc. 74 Inverness Drive East, Englewood, CO 80112 USA

Sales Office: Getinge USA Sales, LLC 1777 East Henrietta Road, Rochester, NY 14623 USA

www.getinge.com