



# Getinge Assured Bowie-Dick Test Card

Results Comparison

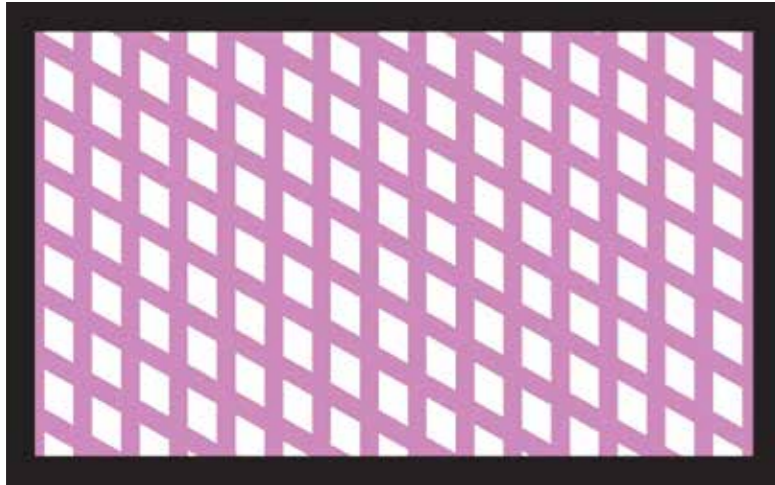


**Unexposed**

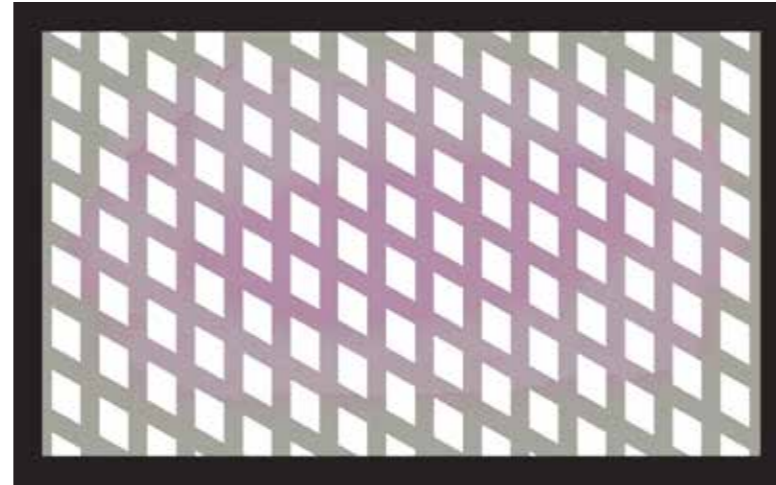


**Fail**

Any remaining purple indicates a fail

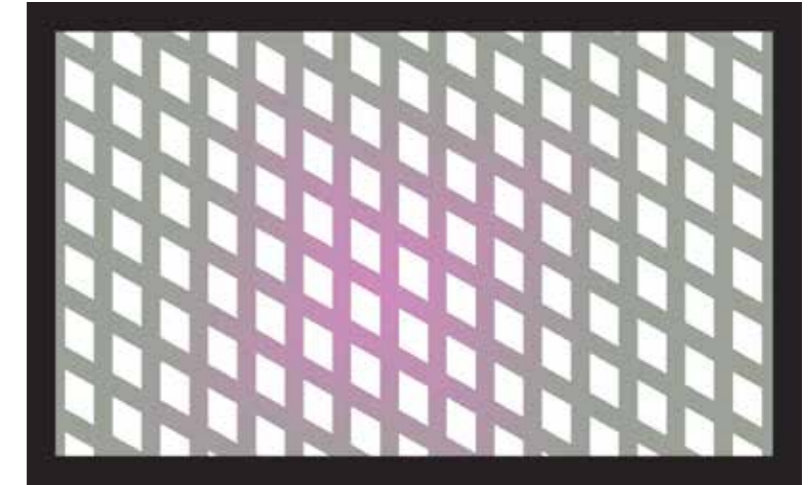


**Fail**

Any remaining purple indicates a fail

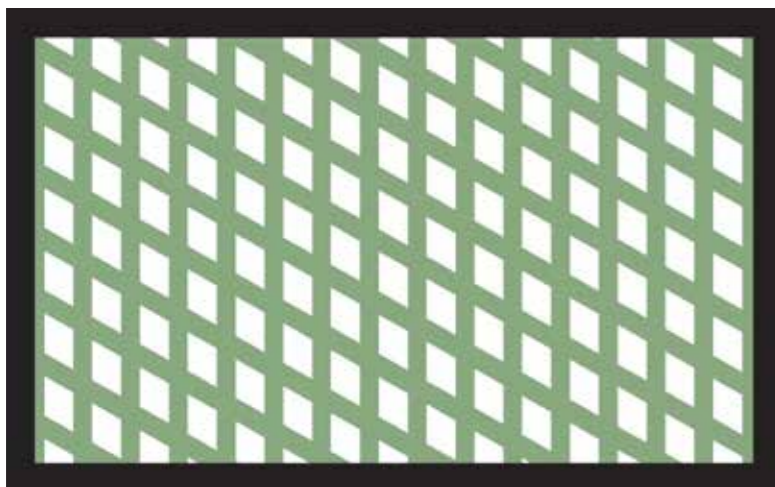


**Pass**

Uniform color change

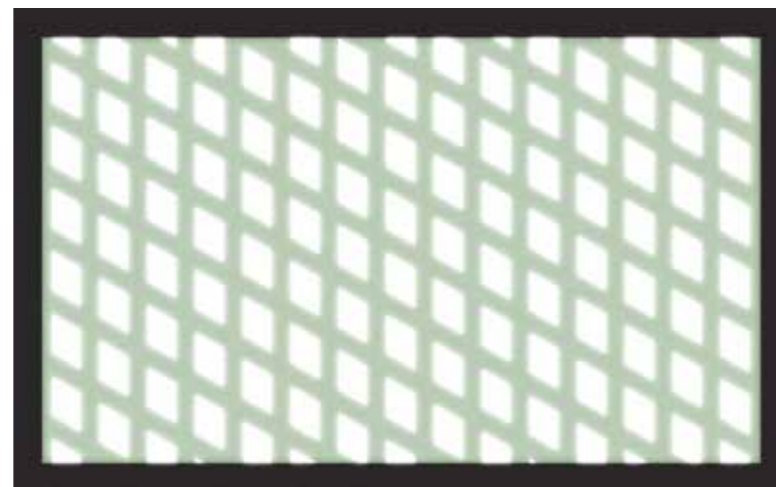

**Pass**

Uniform color change  
(excess condensate)

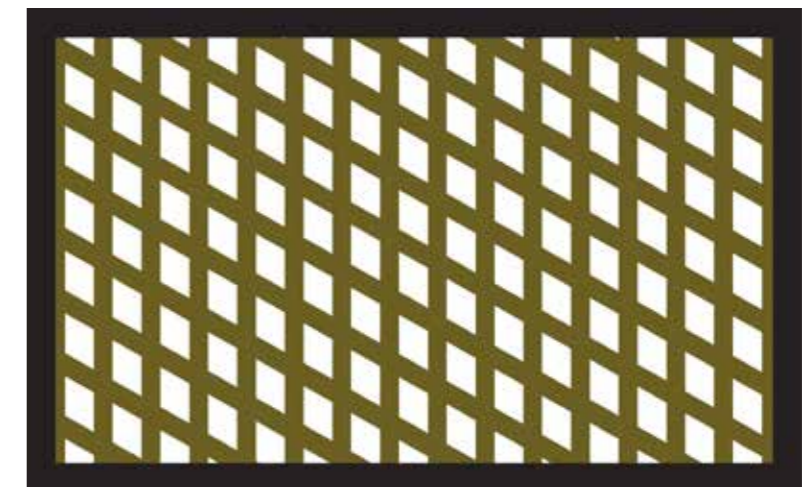


**Pass**

Uniform color change  
(Overexposure/Excess dry time)

# Getinge Assured Bowie-Dick Test Card

Results Comparison

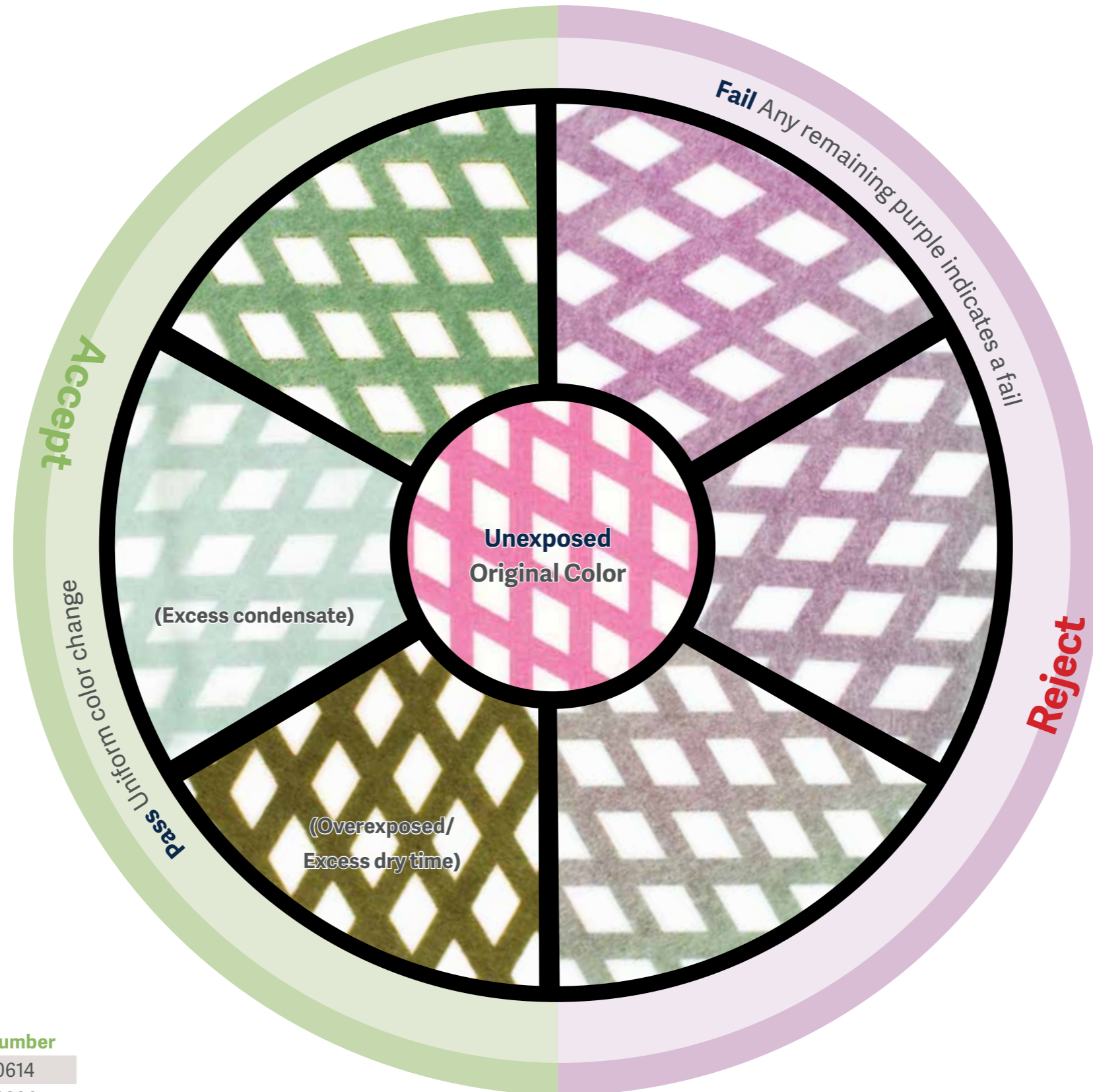

## Standards Compliance

ANSI/AAMI/ISO 11140-5

## Ordering Information

Getinge Assured	Order Number
Bowie-Dick Test Card (30/Bag)	6005500614
Bowie-Dick Test Card Holder (1/Box)	6005500396

Getinge is a registered trademark of Getinge AB or its subsidiaries or affiliates in the United States or other countries.  
Copyright 2025 Getinge AB or its subsidiaries or affiliates. • All rights reserved.

Sales Office, Healthmark Industries Co., • 18600 Malyn Blvd. • Fraser, MI 48026

[www.getinge.com](http://www.getinge.com)