


Getinge Assured

Bowie-Dick Test Card

The Getinge Assured Bowie-Dick Test Card is designed for daily monitoring of pre-vacuum steam sterilizers operating at 132°C for 3.5 minutes to detect the presence of residual air.

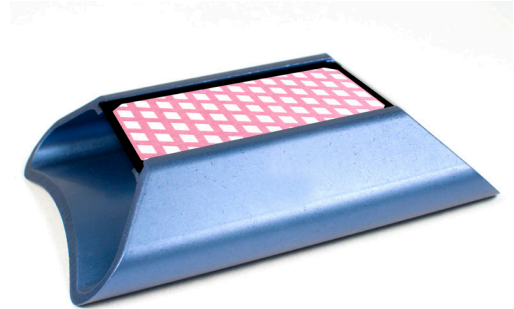

Performance Characteristics

If no significant amount of residual air is present, the indicator will show a uniform green color change. If the indicator shows a purple color in the center, this indicates the presence of residual air.

Key Features

- Complete color change - 3.5 minutes at 132°C
- Re-usable holder keep card in place
- No disposable paper pack

Storage

The storage conditions are printed on the product packaging. Protect from moisture and humidity. Store in resealable bag provided.

Shelf Life

18 months from date of manufacture. The expiry date is printed on the product packaging.

Standards Classification

- FDA Cleared Air Removal Indicator
- ANSI/AAMI/ISO 11140-5

Ordering Information

Getinge Assured	Order Number
Getinge Assured Bowie-Dick Test Card	6005500614
Getinge Assured Bowie-Dick Test Card Holder	6005500396

Getinge is a registered trademark of Getinge AB or its subsidiaries or affiliates in the United States or other countries. Copyright 2025 Getinge AB or its subsidiaries or affiliates. • All rights reserved.

Manufactured by: Getinge Sterility Assurance, 74 Inverness Drive East, Englewood, CO 80112 USA