



Getinge Assured Mini Bowie-Dick Test

Results Comparison

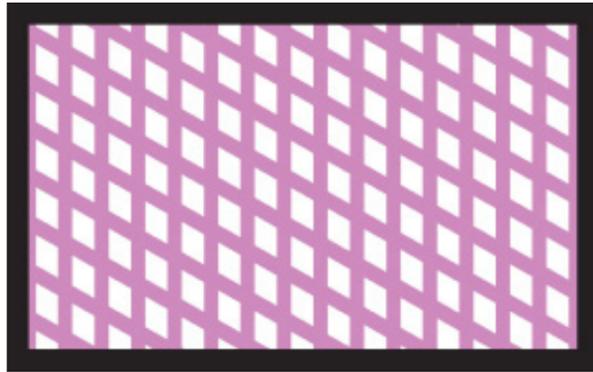


Unexposed

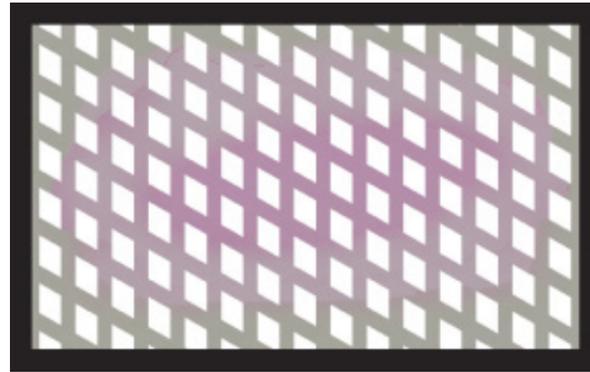


Fail

Any remaining purple indicates a fail

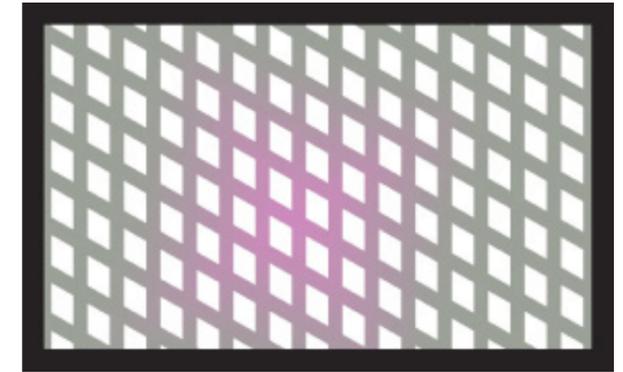


Fail

Any remaining purple indicates a fail

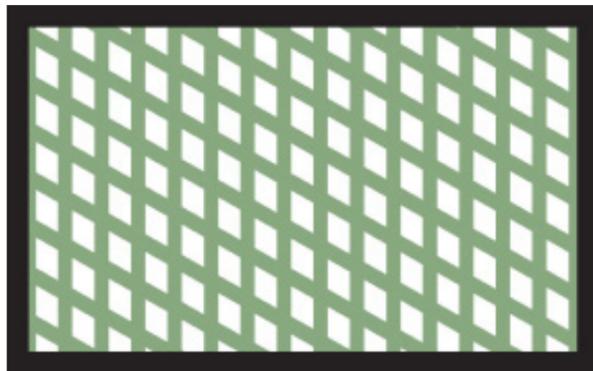


Pass

Uniform color change

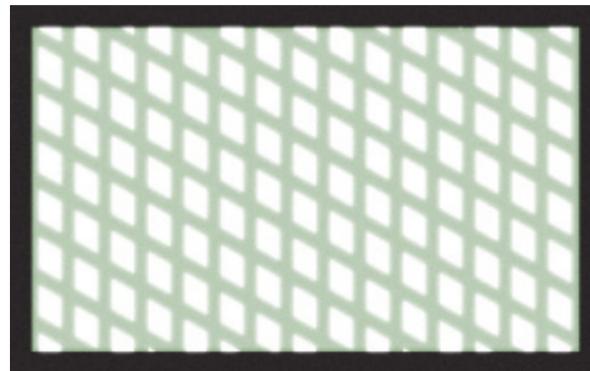

Pass

Uniform color change
(excess condensate)

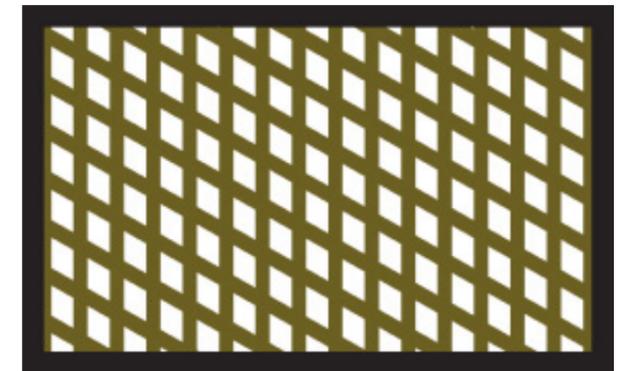

Pass

Uniform color change
(Overexposure/Excess dry time)

Lead and toxic heavy metals have not been used in the making of the indicator test sheet.

Standards Compliance

ANSI/AAMI/ISO 11140-5 and ANSI/AAMI/ISO 11140-1:2014, Type 2

Ordering Information

Getinge Assured	Order Number
Mini Bowie-Dick Test Pack (30/Case)	61301606667



Getinge is a registered trademark of Getinge AB or its subsidiaries or affiliates in the United States or other countries. Copyright 2020 Getinge AB or its subsidiaries or affiliates. All rights reserved.

Sales Office, US • 45 Barbour Pond Drive • Wayne, NJ 07470
Sales Office, Canada • 90 Matheson Blvd West, Suite 300 • Mississauga, Ontario L5R 3R3 • Canada
www.getinge.com