


<b>Brand Name of Product</b>	Face Shield with Drape
<b>Product Code Number(s)</b>	SHL-005
<b>Intended Use</b>	Intended to protect personnel in device reprocessing
<b>Range of Applications for Product</b>	For decontamination use during various sink activities, such as rinsing, flushing, brushing.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Face shield dimensions: 9x13 inches (228x330 mm)</li> <li>• Drape dimensions: 12x12 inches (300x300 mm)</li> <li>• Alcohol repellent and anti-static SMMS drape</li> <li>• 10 mil face shield</li> <li>• Anti-static face shield</li> <li>• Anti-fog face shield</li> <li>• Not made with natural rubber latex</li> <li>• One (1) inch thick brow foam</li> <li>• Protective film on shield with easy removal tabs</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	No special storage conditions
<b>Packaging Contents</b>	100 pieces/case
<b>Shelf Life</b>	N/A

<b>Instructions for Using Product</b>	
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Remove from packaging</li> <li>2. Use easy removal tabs to peel protective film off of shield front and back</li> <li>3. Place foam against forehead</li> <li>4. Secure the elastic band around the back of head</li> <li>5. Tuck the drape into torso garment so that it covers the front neck area</li> <li>6. Adjust as needed</li> </ol>
<b>Special Warnings and Cautions</b>	<ul style="list-style-type: none"> <li>• This face shield is to be used in accordance with your facility's policies and procedures</li> <li>• This face shield should not be in the presence of a high intensity heat source or flammable gas</li> <li>• Healthcare facilities are responsible for decisions regarding appropriate PPE for their personnel</li> </ul>
<b>Disposal</b>	Single use

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	N/A
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A
<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	N/A
<b>Storage</b>	N/A
<b>Additional Information</b>	N/A
<b>Related Healthmark Products</b>	
<b>Other Product Support Documents</b>	PPE Brochure
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com